silicone therapy for scars

Scar treatment

Silicone dressing

silicone dressings

Mamma Anchor
4x30 cm
1,5x12"

Mamma
4x30 cm
1,5x12"

Abdominal
4x40 cm
1,5x16"

Keyhole
10x18 cm
4x7"

Nipple
10 cm
4"

5x20 cm
2x8"

40x100 cm
16x39,5"

15x20 cm
6x8"

10x15 cm
4x6"

40x40 cm
16x16"

BAP Medical BV, 7324 BT, Laan Van De Maagd 119, Apeldoorn.

Produced exclusively for BAP Medical by M-technologies, France.

EN

Name:
BAPSCARCARE S silicone dressing. Self-adhesive, thick, washable and transparent medical silicone dressing for the treatment and prevention of abnormal (hypertrophic and keloid) scars.

Product description:
BAPSCARCARE S is a range of silicone dressings for the treatment of both old and new scars and more extensive scars. Because of its flexibility, the BAPSCARCARE S self-adhesive dressing stays in place on any part of the body. BAPSCARCARE S is washable and can be used under clothing and with compression therapy.

Composition:
Polyurethane film / medical silicone gel

Properties:
BAPSCARCARE S silicone dressings make scars flatter, more supple and thinner and help reduce symptoms (persistent redness, pulling, itching, restricted mobility at joints, discomfort and cosmetic complaints). The dressing provides a covering, occlusive environment on top of the skin, resulting in increased hydration of the skin (epidermis). The increased hydration helps the damaged scar tissue heal more quickly and reduces functional complaints.

Indications/uses:
BAPSCARCARE S silicone dressings can be used on both old and new scars:
- Treatment and prevention of abnormal scars (hypertrophic and keloid)
- Treatment of burn scars
- Treatment of surgical scars
- Persistent functional complaints (persistent redness, pulling, itching, restricted mobility at joints, discomfort and cosmetic complaints)

How to use:
Building up the treatment
We recommend building up the use of the treatment to 24 hours a day during the first week to allow the skin to get used to the treatment gradually. For the best possible result, the silicone dressing should be worn for as long as possible each day.

Before use
Clean and dry the skin and around the scar thoroughly. Remove the product from the pack. If necessary, the silicone dressing can be cut to size. Make sure the dressing covers the entire scar and approx. 2 cm of the healthy skin around the scar. Hair removal around the scar prior to the application increases the adhesion and the durability of the silicone dressing.

Application
Remove the first part of the protective film on the silicone dressing and apply the self-adhesive silicone dressing carefully to the beginning of the scar. Remove the second part of the protective film and apply the next part of the silicone dressing to the scar. Depending on the size of the dressing, repeat the above until the entire dressing has been applied. If necessary, BAPSCARCARE S can be secured with a fixation- or stretch bandage.

Maintenance:
- Should the dressing loses its adhesion, it can easily be washed with a neutral soap/lotion
- After cleaning, rinse the dressing thoroughly using lukewarm, running water
- The adhesive properties will further decrease once the dressing comes in contact with water
- Leave the dressing to dry on a smooth surface, adhesive side up
- During the drying process, the dressing's adhesive properties will gradually return
- Use the dressing again after it has dried completely

How long can I go on using BAPSCARCARE S?:
You can use BAPSCARCARE S for as long as the scar is raised, red or gives rise to symptoms. Ask your doctor for a precise treatment plan.

Duration of use:
BAPSCARCARE S is available in packs of 1, 2 or 4 pieces. One dressing can, with the right maintenance, be used for several weeks.

Special care and precautions:
- Consult your doctor in the event of skin irritations or skin problems.
- The silicone dressings are for external use only.
- Only use the silicone dressings on closed skin: never use on open wounds.
- Do not dry the silicone dressing in the sun or close to a heating source.

Side-effects:
No side-effects have been reported to date.

Storage:
- See pack for expiry date.
- Store at temperatures not exceeding 25°C.

Date of last revision of instructions for use:
November 2016

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